

Toxicology Urine Sample Checklist



Security Tape

- ✓ Collector's Signature present
- ✓ Specimen Collection Date present
- ✓ Donor's Signature present
- ✓ Secured over top of tube
- ✓ Does not cover barcode label
- ✓ Does not cover tube EXP. date

Sample Volume

- ✓ Filled until vacuum stops
- ✓ Full tube near label top
- ✓ A little empty space OK

Barcode Label

- ✓ Correct Donor Name, Date of Birth, and ID#
- ✓ Correct label type states
"TOX Secure Yellow V9"
- ✓ Label in line with tube
- ✓ All parts of label visible
- ✓ Does not cover tube EXP. Date

"A reliable result begins with a good supervised collection which includes an accurately identified and clearly labeled donor sample."