

# Contents of Container



Cardboard Outer Box Container



Styrofoam Inserts



Koolit Freezer Packs (Freeze before Use)



Two Ziploc bags with absorbent material



Blue Pad placed at bottom of Container

# Packing Instructions

When shipping specimens to CDD using the new container box with Styrofoam insert:

1. Place specimens in small Ziploc bags using the absorbent pads. Wrap the closed Ziploc tightly around the specimens.



2. For refrigerated specimens, insert **FROZEN** Koolit Freezer Pack, as shown below, and then place the bagged specimens in the compartment.
3. For all other specimens, place the bagged specimens in the compartment with no ice pack.



4. Place lid over specimens and place inside Container



# Packing Instructions

5. Close the outer container lid.



6. Secure outer container with rubber band.



7. Place container box in Clinical Lab Pack Bag.



8. Attach CDD return label in designated place.

